on

Intended use. Heparinized capillary tubes intended for blood sampling.

- WARNING - Risk of infection** Only left authorized personnel collect and work with blood samples. Make sure gloves are worn.

WARNING - Risk of erroneous high cK results

During sample handling, cks in viscous blood may break down. This may result in falsely high cK results.

- CLSI document GP55-A2, Clinical Laboratory Waste Management

Sampling procedure using mixing wires and end caps

- Loosey put an end cap on one end of the capillary tube.
- Put a mixing wire into the capillary tube and let it slide to the same end as the end cap.
- Select the sampling site and increase the circulation in the region.
- Apply antiseptic to the sampling site and let it dry completely prior to collection.
- Puncture the skin so that drops of blood form quickly.
- Fill the capillary tube with blood from the middle of the blood drop so that air does not enter the capillary tube.
- Tighten the end cap. Put an end cap on the other end of the capillary tube and tighten the end cap.
- Place the capillary tube to mix the blood and anticoagulant.

Note: For best results, make sure that the mixing wire moves from one end cap to the other a minimum of 20 times. If the wire does not move, gently use the magnet to move it along the capillary tube.

For best results, analyze the sample immediately. If storage is unavoidable, store the capillary tube horizontally at room temperature and analyze within 10 minutes.

Puncture the skin so that drops of blood form quickly.

Fill the capillary tube with blood from the middle of the blood drop so that air does not enter the capillary tube.

Tighten the end cap. Put an end cap on the other end of the capillary tube and tighten the end cap.

Place the capillary tube to mix the blood and anticoagulant.

Note: For best results, make sure that the mixing wire moves from one end cap to the other a minimum of 20 times. If the wire does not move, gently use the magnet to move it along the capillary tube.

If storage is unavoidable, store the capillary tube horizontally at room temperature and analyze within 10 minutes.

Puncture the skin so that drops of blood form quickly.

Fill the capillary tube with blood from the middle of the blood drop so that air does not enter the capillary tube.

Tighten the end cap. Put an end cap on the other end of the capillary tube and tighten the end cap.

Place the capillary tube to mix the blood and anticoagulant.

Note: For best results, make sure that the mixing wire moves from one end cap to the other a minimum of 20 times. If the wire does not move, gently use the magnet to move it along the capillary tube.

If storage is unavoidable, store the capillary tube horizontally at room temperature and analyze within 10 minutes.

Puncture the skin so that drops of blood form quickly.

Fill the capillary tube with blood from the middle of the blood drop so that air does not enter the capillary tube.

Tighten the end cap. Put an end cap on the other end of the capillary tube and tighten the end cap.

Place the capillary tube to mix the blood and anticoagulant.

WARNING - Risk of infection Discard the waste as infectious waste.

CLSI document GP55-A2, Clinical Laboratory Waste Management.

WARNING - Risk of infection and incorrect results

Reuse of single-use devices may lead to infection of patients and erroneous results.

Report to the manufacturer

If you receive this device, or as a result of its use, a serious incident has occurred, please report it to the manufacturer and the competent authority in the Member State.

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CLSI document GP55-A2, Clinical Laboratory Waste Management

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